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Basic cleaning 

Nettoyage de base 

ACHTUNG

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ATTENTION

Before initial use and sterilization of the instruments, a basic cleaning has to be effected as follows:

- ▶ Disassembly of dismountable instruments.
- ▶ Entire, bubble-free insertion in an appropriate detergent. We recommend a commercial dishwashing liquid, taking into consideration the manufacturer's information.
- ▶ Thorough mechanical cleaning of the entire instrument surface using a smooth plastic brush with nylon bristles (REF 2600-60 packing unit 3 pieces).
- ▶ Rinse the instruments finally in desalinated (demineralized) water in order to avoid spots, residues and corrosion on the instruments.
- ▶ Thorough drying of the instruments before sterilization.
- ▶ Before initial cleaning, please read the attached instruction for the preparation of resterilizable instruments acc. to DIN EN ISO 17664.

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Dear Customer.

The new EU Regulation on Medical Devices (Medical Device Regulation) leads to many new requirements. An important new requirement is the information on the packaging label. For this reason, we have updated our label and would like to introduce it to you briefly.

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A. Schweickhardt GmbH & Co. KG · Tel.: +49 7464 98910-700 · Fax: -719 · contact@schwert.com · www.schwert.com