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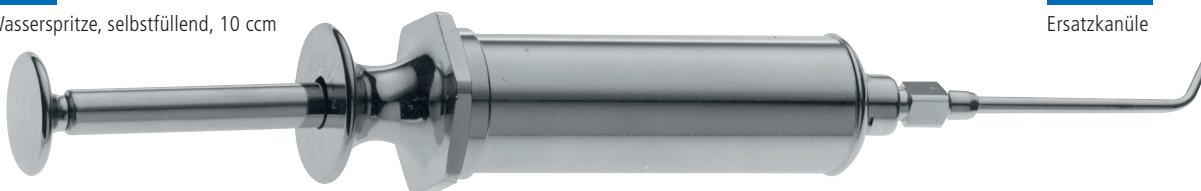
Maintenance Instructions

Dear customer!

Thank you very much for choosing this high quality SCHWERT instrument.
Please find below some tips for maintenance.

IMPORTANT!

- The water syringe has to be cleaned and sterilized before use.
- Please make sure that the syringes are only used by authorized persons.
- Cleaning and sterilization: After use with a patient and before further use, the syringes have to be disassembled, cleaned and sterilized (in the autoclave at max. 134° C / 273° F).

Components of the Self-Filling Water Syringe:

- Self-filling water syringe
- Luer Lock cannula REF **2702-09** spare part
- O-ring REF **2716-2** spare part

Assembly:

- Put the O-ring on to the shank ► scrawl the O-ring afterwards (see pic. 1)
- Screw shank and shell together ► hold shank rod with the O-ring slightly angulated to allow insertion (see pic. 2)
- After putting the cannula on the front part the syringe is ready to use
- To avoid stiffness, it's necessary to lubricate the O-ring before the first use and after each disinfection and before sterilization by medical white oil / instrument oil (REF **3297**).

2716-2
O-ring

3297

50 ml
Paraffinum Perliquidum
(Paraffin oil / Medical white oil)

Important:

- Remove obvious dirt particles from the syringes immediately after utilization. It is not allowed to use fixating detergents or hot water (> 40° C / 104° F) for cleaning as these detergents lock residues and block an effective cleaning process.
Attention: Maximum exposure time: 15 min.!

- **Manual pre-cleaning / cleaning:** In disassembled condition. Drill holes and thread turns must be flushed for at least 10 sec. at a pressure of 3.8 bars by a water pistol.
- **Cleaning:** Please consider our "Instruction for Use according to DIN EN ISO 17664:2004".
- **Drying:** Hollow spaces of the syringes must be dried by sterile compressed air.

2716

Water syringe, self-filling, 10 ccm

2702-09

Spare cannula