Komposit-Instrumente

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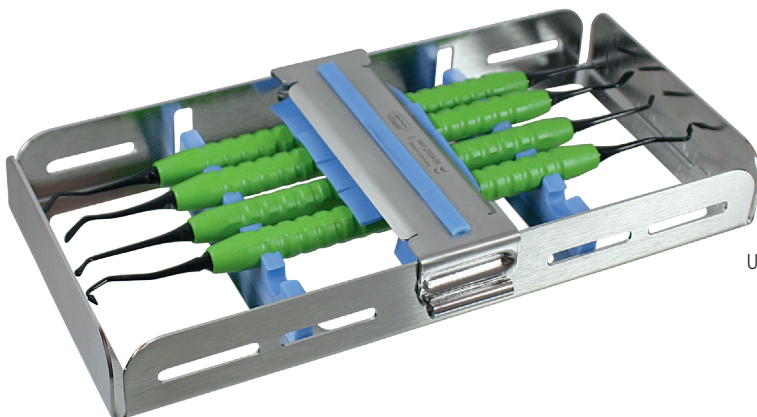
Maintenance Instructions

Dear customer,

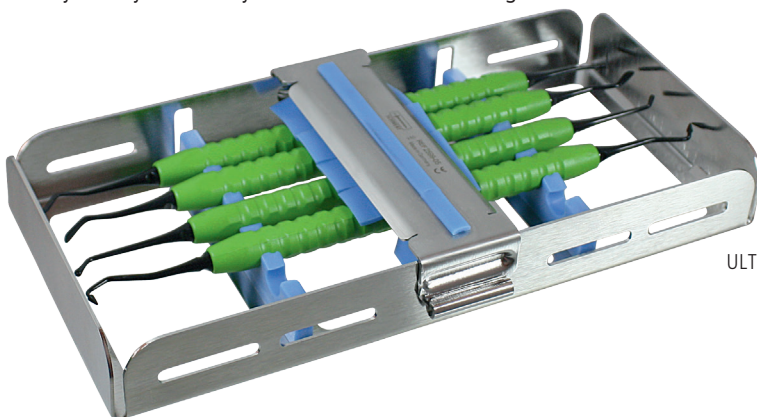
Thank you for choosing these high quality SCHWERT instrument.

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version. See document "0703091007_REPROCESSING_AS_BA_e", downloadable from our website www.schwert.com.

To maximize the life of your Composite Instruments, please follow the instructions below:

- Any contact of the instruments with amalgam has to be avoided (e.g. when inlays are placed or due to residues on gloves).
- Remove residues of material on the instrument immediately – still in the dental chair, e. g. by wiping with a cloth soaked in alcohol, in order to avoid drying-up. Thus, you avoid subsequent manual effort which might possibly lead to scratches on the instrument surface.
- Prior to sterilization, dry the instruments carefully with a clean and soft cotton or microfiber cloth in order to avoid water spots.
- Do not use stainless steel brushes, brass brushes or abrasives for cleaning. To remove tenacious dirt, only use a firm toothbrush. Absolutely avoid any rough mechanical stress (abrasive detergents, e.g. detergent pads, cleaning cloths soiled by foreign particles, etc.).
- Avoid any aggressive cleaning agents (extremely acidic / basic, abrasive effect).
- If an ultrasonic cleaner is used, do not exceed the manufacturer's recommended periods.
- After cleaning, rinse the instruments with clear or even better with demineralized water.
- If the instruments are cleaned in a disinfection bath, take care that the instruments' surfaces are not damaged by other instruments. A contact of the instrument parts during the cleaning process / during the sterilization has to be avoided (danger of contact corrosion).
- In thermal disinfection, we recommend to prepare these instruments in a washtray (see our sets 4085-00Ti or tray 2599-05) in order to avoid damages on the surface.
- The instrument set (4085-00Ti) may be sealed in sterile film in which they are able to be stored up to 6 weeks (acc. to DIN 58953 Part 7).

Thank you very much for your attention and following these instructions.

ULTRA

4085-00Ti

ULTRA Composite set, consisting of: 4085-01Ti, 4085-02Ti, 4085-03Ti, 4085-04Ti, washtray 2599-05