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Dear Customer!

With the purchase of these SCHWERT MICRO mirrors you have chosen a high quality product, the correct handling and use of which are described in the following.

In order to keep risks and unnecessary burdening of the patient, the user and third parties as low as possible, we request that you carefully look through these instructions for use and keep them for further reference.

INSTRUCTION OF USE

Apply a cofferdam to the patient, before using the MICRO mirror, to exclude any danger of aspiration or swallowing of the mirror glass. The mirror glass is adhered with an adhesive into the casing and the MICRO mirror can be sterilized up to 180° C.


The MICRO mirrors are tested by ultrasonic. Despite that, it could – because of small dimensions – perhaps happen, that the adhesion reduces. But this cannot be noticed by our final quality control.

Check the MICRO mirror on completeness when you have finished patient's treatment.


Product is delivered not sterile, has to be sterilized therefore before initial use.

CLEANING AND HANDLING


Because of the small dimensions the strength of glass is only 0.3 mm. Therefore the handling and especially the cleaning has to be done with appropriate circumspection. Pressure / strength can damage the glass. Please avoid crusts and clean the instrument immediately after the patients treatment by putting into disinfection solvent.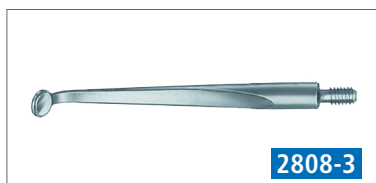

2808-3

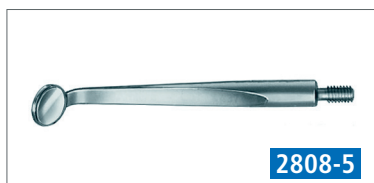2808-5

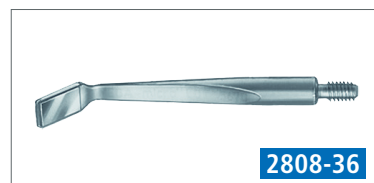2808-36