Mikrochirurgischer Skalpellgriff, Titan, 13 cm

REF 1070-00

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DEMONTAGE:

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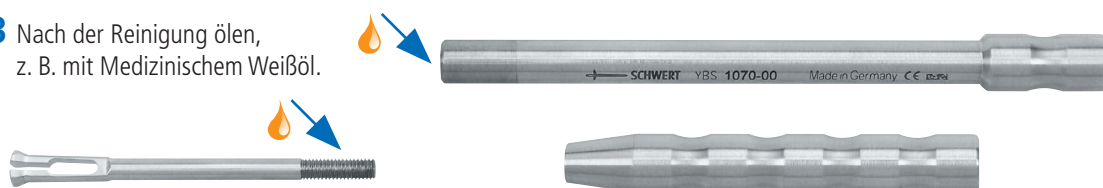
Maintenance and re-processing

DISMANTLING:

The Microsurgical Blade Holder has to be dismantled before cleaning, as shown below.

CLEANING:

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664".

See document "0703091007_REPROCESSING_AS_BA_e", downloadable from our website www.schwert.com.

Pre-Treatment: Use cleaning brush with nylon bristles when incrustations are heavy.

Cleaning: Manual or mechanical – **Attention:** Cleaning in an thermal disinfectant: standing vertically, thread downwards.

Rinsing: Please use deionized water to avoid water spots.

Drying: Use compressed air or air-dry (manual preparation).

MAINTENANCE:

Oiling with Medical White Oil, as shown.

ASSEMBLY:

Assembly is made in reverse order to the dismantling.

3297 50 ml

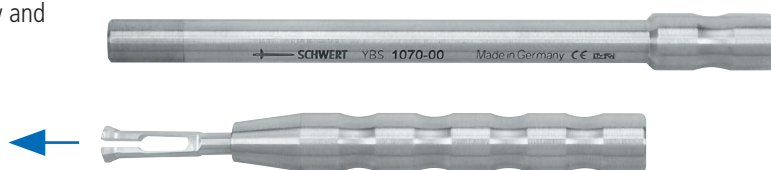
Paraffinum Perliquidum
 (Paraffin Oil / Medical White Oil)

For maintenance of instrument hinges and for use as sharpening oil. Physically harmless to DAB 10.


1 Turn off the back end by twisting the handle counterclockwise.

2 Remove the back end completely and pull-off the insert to the front.

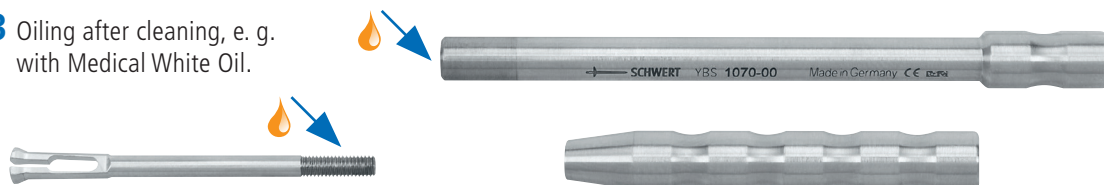3 Oiling after cleaning, e. g. with Medical White Oil.

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