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SCHWERT
SWORD · ESPADA
EPEE · SPADA

SCHWERT Bone Mill REF 1720

Assembling & Disassembling Instructions

Dear valued customer!

You have decided for a high quality product from our company – many thanks. In order to ensure your long-term satisfaction, we would like to explain the use of the SCHWERT Bone Mill as follows:

Application

For crushing the bone material the handle has to be turned with appropriate effort. The result of the granulate can be influenced through various pressure on the pestle (direction of rotation see photo).

After using, the SCHWERT Bone Mill can be dismantled completely within four steps (without tools) and the sterilization and cleaning processes can be made.

Cleaning and reprocessing

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version. See document "0703091007_WIEDERAUFBEREITUNG_AS_BA", downloadable from our website www.schwert.com.

Guarantee

A. Schweickhardt GmbH & Co. KG supplies only tested and defect-free products to its customers. All of our products are designed and manufactured to comply with the most stringent quality requirements. We assume no liability for products that have been altered compared to the original, used for purposes for which they were not intended or used improperly. Products that are repaired or altered at any company other than SCHWERT or a company commissioned for this purpose by SCHWERT are no longer authorized to bear the CE symbol.

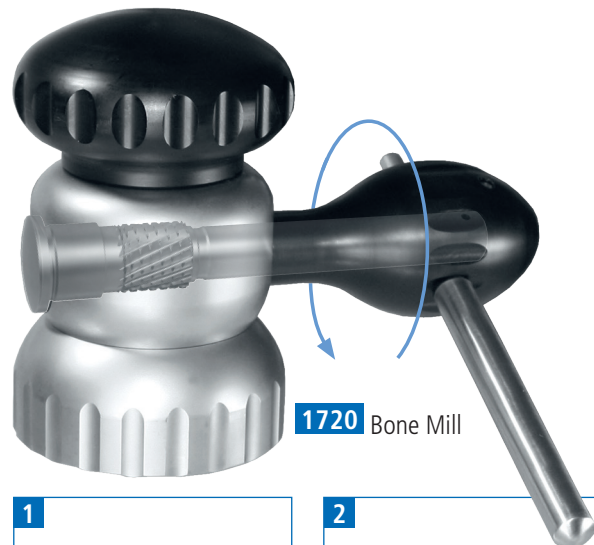
Decontamination information

On the basis of statutory regulations and above all to protect our employees, we need a signed "Evidence of decontamination" with every return.


Please ensure that this "Evidence of decontamination" is enclosed filled out and signed with **every product return** (complaint/repair/other reason for return) and package the product such that there is no risk of injury for our personnel in goods receiving during unpacking.

This form is available for download on our website "www.schwert.com".

Release version – see footer

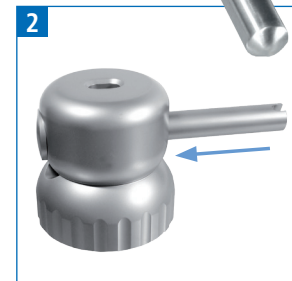Step 1: Take the lid/pestle and pull the handle element from the milling device.

Step 2: Remove the milling device, by pushing it through the filling corpus.

Step 3: Take the filling corpus from the collecting bank.

Step 4: Every single part can now be processed through the sterilizing and cleaning process.

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A. Schweickhardt GmbH & Co. KG · Tel.: +49 7464 98910-700 · Fax: -719 · contact@schwert.com · www.schwert.com